

## Lifetime Limited Warranty

If you have a question about your MOBB Healthcare device or this warranty, please contact an authorized MOBB Healthcare dealer.

MOBB Healthcare warrants to the original consumer purchaser for as long as they own their MOBB product that it will be free from defects in material and workmanship under normal use. This warranty is only applicable to products used in residential applications within the United States and Canada and is not transferable to subsequent owners. Should defects in material or workmanship exist during the applicable warranty period in any MOBB Healthcare Product, MOBB Healthcare will elect at its discretion to either repair or replace the defective material. This warranty covers only parts and materials of the products supplied by MOBB Healthcare. Not covered under this warranty are wear and tear items or abuse of the product in a manner that was not intended for the product to be used. This warranty shall become void if the product is in any way modified, improperly adjusted or abused.

**MHSCMWII**



844604101608



## MOBB Healthcare Ltd.

116A Viceroy Rd. Unit 5-6  
Concord, Ontario, Canada  
L4K 2M1

[www.mobbhhc.com](http://www.mobbhhc.com)



## Steel Commode Chair with Wheels II



## commode assembly instructions and precautions



**FIGURE 1**

### 4 Caster Breaks

## assembling commode

1. Before assembling seat, allow toilet seat to reach room temperature if exposed to cold. This will help prevent the seat clamps from cracking while being assembled onto commode back cross brace.
2. Insert Back support into frame.
3. To drop arm, depress release button on arm and gently allow to drop until reaching arm stop.
4. Align the toilet seat clamps onto the back frame rail. (Both clamps should sit outside the pail brackets). Push down on seat until clamps snap firmly into place.
5. When removing the seat grasp the back of the seat and lift.
6. Install commode bucket by placing in bucket frame. Reverse to remove.

## additional precautions

1. Toilet seat must be in down position before use. Failure to do so may result in breakage of commode and possible injury.
2. Do NOT install commode without first reading and understanding this assembly and instruction sheet. If you can not understand these instructions, contact a healthcare, dealer or technical professional before attempting to install. Otherwise, injury or damage may occur.
3. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
4. Ensure ALL screws, nuts and/or bolts are tightened.
5. Users with limited physical strength should be supervised or assisted while using commode.

## Front Access

Flip Seat for Front Access



## Back Access

Flip Seat for Back Access

