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ÉTAPE UN >>>

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STEP ONE >>

Putting your walker together:

- Upon receiving your walker, please check the contents carefully. Report any missing items to your dealer or Evolution immediately:
- Contents of all Evolution Xpresso walkers are as follows:
 - Folded walker complete with main frame, seat, handles, and wheels already installed.
 - A soft folding basket
 - Back support
 - Warranty Registration Card
 - Owner's Manual
 - Soft Seat Cushion
- Adjust handle height. Put Knob Bolts through from inside. Tighten until it is snug.
Caution: Do Not Overtighten.

STEP TWO >>

Checking the brakes and wheels:

- The Evolution braking system provides safety and ease of operation. Hold (squeeze) to brake and press down on the brake handle for parking.
- Check braking system. The wheels should stay locked while parking brake is activated.
- Do not activate parking brake while walker is moving. This may cause permanent damage to the wheels.
- Always activate parking brake before using the seat.

STEP THREE >>

Maintenance and care of your Xpresso Walker:

- Check tightness of nut and bolts regularly.
- Check frame and tire integrity at least once a month.

Folding, Unfolding, and Carrying your walker:

To FOLD your Xpresso walker, pull on the seat center handle and the walker will remain standing up. To UNFOLD the Xpresso, simply push down on the handle in the middle of the seat making sure the walker locks in the fully open position. **Caution: Ensure the center seat handle is level with the seat before sitting.**

To carry the walker while folded, simply pick it up by the center handle and lift the walker carrying it at your side like a suitcase.

Brakes:

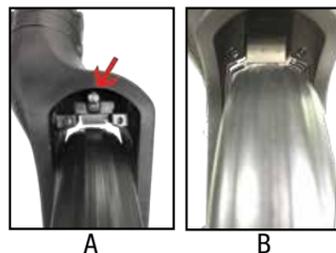
Apply the brakes by pulling both brake handles upwards. Do not walk with brakes on.

Parking Brakes:

Press down on both brake handles until you hear a distinct click sound. Always make sure that both brakes are engaged. Pull upwards to release the parking brakes. When using the seat, both brakes MUST be in the locked position for your safety.

Brake Adjusting:

- To ensure proper braking, from time to time adjust the brake pads located on top of the rear wheels. To adjust the brake, remove the Slow Down Brake, if installed, by removing the 2 Phillips screws on front of the rear fork.


- Using a Phillips screwdriver, rotate the adjustment screw clockwise (A) to lower the brake pad (B) for tighter braking. Test the brakes, making sure that when applying the parking brake, the rear wheels stay in the locked position.
- Reinstall the rear fork cover or Slow Down Brake if installed.

Handle height adjustment:

To adjust the handles to a suitable height, stand behind the walker with your shoulders level. The handles should be at the same height or slightly higher than the user's wrist. A therapist might suggest a different handle height to accommodate a specific need for the user.

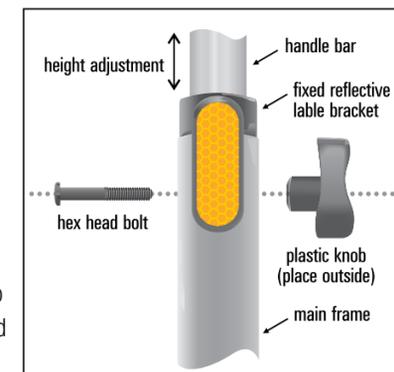To adjust the handles, remove the knob and pull the hex bolt out, adjust the handle to the prescribed or desired handle height and insert the hex bolt ensuring that the hexagonal part of the bolt fits properly on the hexagonal cut out on the frame of the Xpresso and tighten the knob snugly. **Caution: Do Not Over Tighten.**

Installing Basket:

To install the folding soft basket, fold the Xpresso part way by pulling on the center seat handle and insert the basket black brackets on the rails on both inside frames, push on the middle of the seat to open the walker.

Caution: Ensure you hearing the "Click" sound for proper positioned the basket.

Assistance:

Your nearest Evolution dealer will be pleased to assist you with the assembly and/or maintenance of your walker.

Factory Repair Information >>

Evolution's motto is 'Committed to Providing Unsurpassed Quality & Service'. Our goal is to provide you the best service in the industry. If you encounter problems with your Evolution Walker or need help, please:

- Review your documentation:** You will find printed documentation shipped with your Evolution Xpresso Walker. They contain maintenance guides and installation information.

- Call or visit your Evolution dealer:** They will be able to give you the assistance you need and determine if your walker is eligible for warranty.
- Contact Evolution Customer Service at:** 1-800-556-2558 (U.S. and Canada)

For Warranty Service, Repair, or Replacement:

If you have to return your walker to Evolution for warranty service, please call or visit your Evolution dealer and they will be happy to assist you. If you need to locate an Evolution dealer close to you, please contact us at 1-800-556-2558. Be prepared to provide 1) the unit model number, 2) the serial number, and 3) a description of the problem.

For Out-of-Warranty Service or Repair:

If your walker is out of warranty or is not eligible for warranty, your Evolution dealer will be happy to provide estimates to have your walker repaired.

A few things to remember:

If the problems with your walker are deemed to be the result of improper use, such as hitting curbs, hit-opening doors, etc., it will not be covered under warranty. We reserve the right to determine if your walker has been misused. Your Evolution dealer will be happy to assist you in arranging the return of your walker. We will service the walker within 2 working days after receiving it and it will be on its way back. If we cannot service your walker within 2 working days after receipt, we will send you a new one!

Optional Accessories for Extra Convenience >>

Cup Holder >

Cane Holder >

Curb Climber >

Makes getting over curbs and other obstacles much less of a challenge

Slow Down Brake >

Added security and protection for travel on slopes, declines, and hills.

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